


Infant Discharge Cart™ Replacement Caster Installation Instruction

Please Read All Instructions Before Beginning the Installation.

What is Included:

Item 1

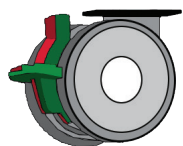

(1) Twin Caster - Brake and Direction Lock

Item 2



(1) Caster Total Lock - Single Brake Peddle

Item 3



(1) Caster

Item 4



(1) Caster Static Dissipative Yellow Dot

Item 5




(4) Hex Head Flange Serrated Bolt 1.5

Item 6




(2) Spacer Blocks

Item 7

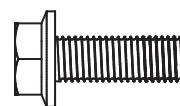

(12) Hex Head Flange Serrated Bolt 1.0

Tools Needed:

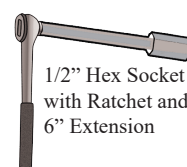

1/2" Hex Socket with Ratchet and 6" Extension

Step 1: Remove old bolts and casters using the ratchet.

Step 2: Replace with new casters and bolts in proper location as shown

Step 3: Tighten all bolts until Caster Plate is fully and firmly seated against plastic body. Do not over tighten.

Note: Caster Item 1 requires Item 6 Spacer Block and Item 5 the longer bolt.

