


Infant Mini Transport™ Replacement Caster Installation Instruction

Please Read All Instructions Before Beginning the Installation.

What is Included:

Item 1

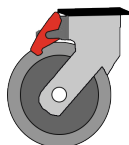

(2) Caster Total Lock
- Single Brake Peddle

Item 2




(1) Caster

Item 3




(1) Caster
Static Dissipative
Yellow Dot

Item 4

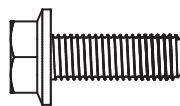

(16) Hex Head Flange
Serrated Bolt 1.0

Tools Needed:

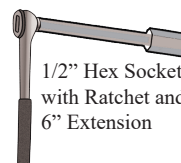

Step 1: Remove old bolts and casters using the ratchet.

Step 2: Replace with new casters and bolts in proper location as shown

Step 3: Tighten all bolts until Caster Plate is fully and firmly seated against plastic body. Do not over tighten.

