



**Patients
First
Products**

Revolutionizing the Patient Experience

Infant Discharge Cart Instructions

The Infant Discharge Cart comes ready to use. No Assembly Required.

User instructions are permanently affixed to the cart.

Step 1:

Place carrier in the car seat holder making sure the infant's angle of incline matches the Infant Carrier Manufacturers recommendation. This is achieved either by the carrier's built in level or a line pictured on the carrier that must be parallel to the floor. Once this established secure using the two hooks.

If you would like a leveling device to check the angle of the infant's recline yourself, we will send one at no charge. Just call 973-885-6779.

Step 2:

Tighten each hook by pulling on the webbing tab.

To release the hooks:

Pull the plastic tabs on the hooks up and move the hooks off of the carrier.



Safety Warnings:

- ★ Never place your infant directly in this holder.
- ★ Always use your own infant carrier and fasten carrier securely.
- ★ Max. 18lbs / 8 kg
- ★ Stay with your infant at all times.
- ★ Do not allow children to ride or climb on Infant Transport.


Place carrier in the car seat holder making sure the infant's angle of incline matches the Infant Carrier Manufacturers recommendation. This is achieved either by the carrier's built in level or a line pictured on the carrier that must be parallel to the floor. Once this established secure using the two hooks.

If you would like a leveling device to check the angle of the infant's recline yourself, we will send one at no charge. Just call 973-885-6779.



To track straight, press green peddle down.

Always press down 2 brakes when loading and unloading.

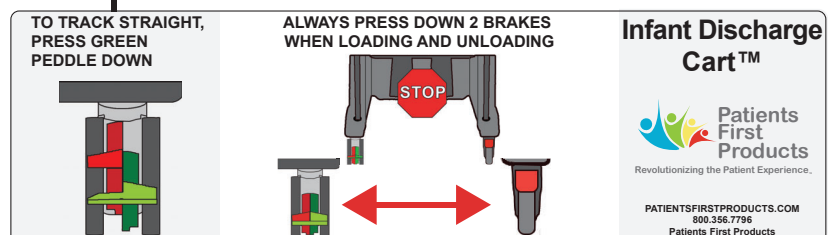

PFPCares.com
973.885.6779



13830 Jetport Commerce Parkway
Fort Meyers, FL 33913